


# Are NovoPen 6® & NovoPen Echo® Plus compatible with my smartphone?

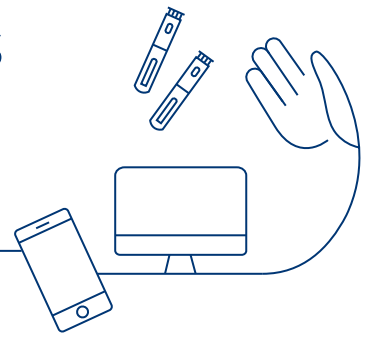

## Here's how to find out:

If you can answer "Yes" to all of the following questions, your smartphone is compatible with NovoPen 6® and NovoPen Echo® Plus

- ✓ Does your smartphone have an NFC (Near Field Communication) interface?
- ✓ Is your smartphone's operating system (OS) compatible with NFC interface?
- ✓ Is the app you prefer supported by your smartphone's operating system?

Depending on the model, the NFC interface is in a different place on your smartphone. You can see some examples here.



iPhone



Samsung



Huawei

The individual specifications of your smartphone can be found in the operating instructions or can be requested from your smartphone dealer.

## NovoPen 6® & NovoPen Echo® Plus are compatible with many popular smartphones

|                 | Model  | Operating system                 |
|-----------------|--|----------------------------------|
| <b>Apple®</b>   | iPhone 8, 8 Plus, X, XR, XS, XS Max, XR, 11, 11 Pro, 11 Pro Max, Pro, SE, 12, 12 Mini, 12 Pro, 12 Pro Max, 13, 13 mini, 13 Pro, 13 Pro Max   | iOS 13 and higher                |
| <b>Samsung®</b> | Galaxy S7 Edge, S8, S8+, S9, S10, S20, Galaxy S20 FE, Galaxy S21, Galaxy A50s, Galaxy A51, Galaxy A52, Galaxy A70s, Galaxy A71, Galaxy A90, Galaxy Note 10+  | Android: 8, 9, 10, 11            |
| <b>Google®</b>  | Pixel, Pixel 2, Pixel 2 XL, Pixel 3  | Android: 8.1.0, 10               |
| <b>LG®</b>      | Nexus 5X   | Android: 8                       |
| <b>Huawei®</b>  | <b>P:</b> P10 Lite, P20, P30, P30 Pro, P40, P40 Pro, P40 Pro+, P50 Pro,<br><b>Mate:</b> 20, 20X, 40, 40E, 40 Pro, 30,<br><b>Honor:</b> 10, X10 Max, 20, 20 Pro, V20, V30, 30, 30 Pro, 30 Pro+<br><b>Nova:</b> 4, 5 Pro, 6, 7, 7 Pro. | Android: 8, 9 10<br>Harmony: 2.0 |

A full dose log, with 800 entries, will take 9-11 seconds for the full data transfer. Moving the pen so it falls outside of the range of the NFC antenna could cause the dose log transfer to be incomplete

## How to find the right app for your Smartphone

Please contact the company / developer of the app you prefer to use with NovoPen 6® & NovoPen Echo® Plus to get an overview of compatible smartphones.