

# Using the NovoFine® Autocover® needle

Attaching the NovoFine® Autocover® needle to your pen device



## Step 1:

Take a new needle. Tear off the paper tab



## Step 2:

Push the needle straight onto the pen. Turn until it is on tight



## Step 3:

Pull off the outer needle cap



## Step 4:

Always check the flow before you begin to inject. This helps to ensure that you get your full dose of medicine



## Step 5:


Inject your medicine by following the guidelines from your doctor or nurse, and the Instructions for Use that comes with NovoFine® Autocover® and your pen



## Step 6:

After injecting, the safety lock indicator will turn red to indicate that the shield is now locked

## Inject into the right part of the skin

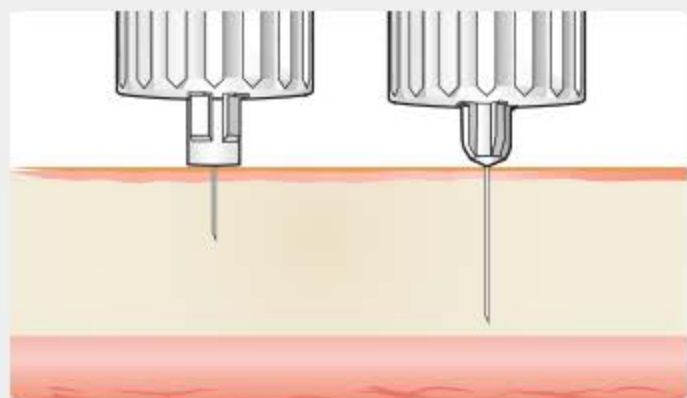

- The medicine must be injected into the fatty layer of the skin called the subcutaneous layer
  - The depth of the subcutaneous layer is different for every person and at different parts of the body
- The length of the needle could influence which injection technique you should use to ensure your medicine is correctly injected
  - Your doctor or nurse will be able to tell you which technique is best for you, and it is best to follow this guidance

## Correct needle disposal

To reduce the risk of injury to others, pen needles should be disposed of in a puncture-resistant container, such as a "sharps container", according to local regulations. Check local recommendations for discarding medical waste.

